

Universal Arm Protector

Instructions For Use

The Universal Arm Protector comes pre-positioned ready to protect patients of average size. To accommodate smaller or larger patients, the Universal Arm Protector can be easily customized by adjusting the component parts (wrap, foam hand/wrist support, and hook tabs).

If you find it necessary to move the foam hand/wrist support, you must use the woven tabs at each end of the foam arm/wrist support to separate it from the wrap. Numbers representing each "step" have been added to The Universal Arm Protector to aid the caregiver during application. Please refer to the corresponding images and steps below.

STEP 1

Make sure to utilize the Arm Protector identified with "1R" on the patient's "right arm" and utilize the Arm Protector identified with "1L" on the patient's "left arm". Ensure that the "1R" or "1L" is facing up.

Open the arm protector by lifting the hook tabs and pulling away to release the two sides

Make sure that you do not completely remove the hook tabs from the surface of the wrap.

STEP 2

With the wrap spread open, position the appropriate arm protector under the patient's arm. The patient's hand should rest on top of the foam hand/wrist support with the palm facing downwards.

Please note: There should be enough room (at minimum 3") between the patient's fingertips and the end of the wrap to pull the end of the wrap back toward the top face of the wrap as referenced in Step 5.

STEP 3

Locate the "3" symbol and pull the wrap over/around the patient's arm and hold in place with your hand.

STEP 4

Locate the "4" symbol and pull the wrap over the "3" symbol/patient's arm. Secure the wrap in place with the white hook tabs.

STEP 5

Fold the top end of the wrap back toward the top face of the wrap and secure the folded material to the top face using the blue hook tab.

STEP 6 (NOT PICTURED)

To access, reposition, or release the patient's arm, release the hook tabs as outlined in Step 5.



Not made with
natural rubber latex



EUROPEAN REPRESENTATIVE

GRI-Alleset Limited B.V.
Kingsfordweg 151, 1043 GR
Amsterdam, THE NETHERLANDS



MANUFACTURER

GRI Medical & Electronic Technology Co., Ltd.
www.alleset.com
1805 Honggao Road, 314031
Jiaxing, Zhejiang, PEOPLE'S REPUBLIC OF CHINA



IMPORTER

GRI-Alleset Limited B.V.
Kingsfordweg 151, 1043 GR
Amsterdam, THE NETHERLANDS

U.S. HEADQUARTERS

Alleset, Inc.
www.alleset.com
4142 Industry Way
Flowery Branch, GA 30542 U.S.A.

