gentleheel[®]

HEEL INCISION DEVICE
FOR NEWBORN BLOOD SAMPLING

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Instructions for Use

EN

Description	CAT#	Blade	Incision Depth	Incision Length
Toddler	TGH10X50	Surgical Steel	2.00 mm	3.00 mm
New Born	NGH10X50	Surgical Steel	1.00 mm	2.50 mm
	NGH4X250			
	NGH5X200			
	PGH10X50	Surgical Steel	0.85 mm	1.75 mm
Preemie	PGH4X250			
	PGH5X200			
Micro-Preemie	MPGH10X50	Surgical Steel	0.65 mm	1.40 mm

MATERIAL REQUIRED

The following materials are required but not provided with gentleheel:

1. Antiseptic per hospital recommendation
2. Sterile gauze pads
3. Blood collection container and capillary tube (or similar device)

CONTRAINDICATIONS

Should not be used on any body part other than the area of the infant heel as identified in the diagram. Should not be used on adults.

WARNINGS

- Follow your facility's recommended procedures for infant blood sampling.
- Follow your facility's recommended procedures for wound care management. It is not advisable to apply an adhesive bandage over the incision site of an infant's heel as the adhesive bandage may cause skin irritation or may be aspirated by the infant.
- Do not use if the sterile package is damaged or open.
- Individuals using this device should be trained in blood collection and trained to use this product.
- Follow your facility's recommended policy for disposing of contaminated sharps.
- In the event of a patient injury related to the device please report the details to the manufacturer and the competent authority of the Member State in which the user and/or patient is located

Shaded areas on heel represent recommended incision sites.

Procedure Detail

Please review and utilize the following steps when using gentleheel.

Step 1: Position and Site Prep

With the infant in a supine position, cleanse the incision site per your facility's recommended procedure for skin prep associated with infant blood sampling.

Step 2: Remove Packaging

Remove the appropriate gentleheel from its packaging.

Step 3: Remove Trigger Lock

Remove the trigger lock. Once the trigger lock is removed, do not depress the trigger or place anything in front of the opening at the base of the device.

Step 4: Incision

Place the opening (at the base of the device) against the cleansed incision site ensuring that the base of the device is flush with the infant's skin. After depressing the trigger, immediately remove the device.

Step 5: Remove 1st Drop of Blood

The first drop of blood should be wiped away from the incision site with a dry sterile gauze pad.

Step 6: Collect Sample

A second drop of blood should form over the incision site. Collect the blood sample with a capillary tube (or similar device). Fill blood collection container until the required volume is obtained.

Step 7: Compression

After the sample has been collected from the infant's heel, the foot should be elevated above the body. Press a dry sterile gauze pad against the incision site until the bleeding stops.

Step 8: Disposal

Dispose of the device in accordance with facility policy related to sharps disposal. (Not pictured)

INTENDED PURPOSE	INTENDED USERS	USE ENVIRONMENT
------------------	----------------	-----------------

The purpose of the gentleheel is to provide a method of lancing the heel area of new born babies while protecting the clinician from accidental sticks after the baby has been lanced.	The intended users of this device will be phlebotomists, lab technicians, nurses, mid-wives and other clinicians who may be involved with blood sampling of new born babies.	This device will be used primarily in new born nurseries, NICU, delivery suites, and laboratories.
--	--	--

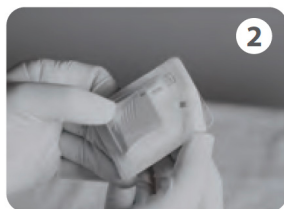
INTENDED PATIENT POPULATION	INDICATIONS	INTENDED USE STATEMENT
-----------------------------	-------------	------------------------

The intended patient population for this device is new born babies without consideration for ethnicity. For premature or small babies a smaller size unit may be required as determined by the clinician.	Indications for use will be determined by the clinician where a new born blood sample is required. The size of the device should be considered based upon the size of the baby as noted by the clinician and the estimated thickness of the fat pad over the heel area.	The intended use of this device is for newborn babies and toddlers of any ethnicity where a blood sample is desired and a self-retracting safety feature is desired to protect the clinician performing the blood sampling. The clinician will determine the size device appropriate to the size of the child and the body fat layer over the heel area.
---	---	--

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Technology Co., Ltd.
Emergo Europe B.V.
Alleset, Inc.